

STERILISATION PROCEDURES SILICONE PROSTHESIS 173514 et al.

Autoclave using one of the following methodes:

1) **Steriliser type:** Gravity
Minimum Temperature: 121 °C
Full Cycle Time: 60 Minutes
Minimum Dry Time: 0 Minutes
Sample Configuration: Sterilisation Pouch

2) **Steriliser type:** Pre-Vacuum
Preconditioning Pulses: 3
Minimum Temperature: 132 °C
Full Cycle Time: 4 Minutes
Minimum Dry Time: 0 Minutes
Sample Configuration: Sterilisation Pouch

“
**THOROUGH CLEANING
RELIABLE RESULTS**
”

EICKEMEYER®

GERMANY | EltstraÙe 8 | 78532 Tuttlingen | Deutschland | T +49 7461 96 580 0 | F +49 7461 96 580 90 | E info@eickemeyer.de | www.eickemeyer.de

DENMARK | Lysbjergvej 6, Hammelev | 6500 Vojens | Danmark | T +45 7020 5019 | F +45 7353 5019 | E info@eickemeyer.dk | www.eickemeyer.dk

UNITED KINGDOM | 3 Windmill Business | Village Brooklands Close | Sunbury-on-Thames | Surrey | TW16 7DY | UK | T +44 20 8891 2007 | www.eickemeyer.co.uk

ITALY | Via G. Verdi, 8 | 65015 Montesilvano (PE) | Italia | T +39 085 935 4078 | F +39 085 935 9471 | E info@eickemeyer.it | www.eickemeyer.it

SWITZERLAND | Sandgrube 29 | 9050 Appenzell | Schweiz | T +41 71 788 23 13 | F +41 71 788 23 14 | E info@eickemeyer.ch | www.eickemeyer.ch

NETHERLANDS | Bedrijventerrein, Pavijen-West Bellweg 44 | 4104 BJ Culemborg | Nederland | T +31 345 58 9400 | E info@eickemeyer.nl | www.eickemeyer.nl

POLAND | Al. Jana Pawła II 27 | 00-867 Warszawa | Polska | T +48 22 185 55 76 | F +48 22 185 59 40 | E info@eickemeyer.pl | www.eickemeyer.pl

CANADA | 250 Briarhill Dr. Stratford | Ont. Canada N5A 7S2 | T +1 519 273 5558 | F +1 780 963 5428 | E info@eickemeyer.com | www.eickemeyerveterinary.com