

MELAG



MADE IN GERMANY



Vet

# SteriHero

My autoclave. My **superpower**.

# Your new **superpower**

No matter what big tasks the day holds in store for you, **SteriHero Vet** has your back. The brand new stand-alone Class B and S autoclave now complies with medical device regulations and is perfectly suited for veterinarians: From the features to the price! Trendsetting technologies, an incredibly advanced display and the unique MELAG quality - made in Germany give you the superpower to change everything:

Now also available as  
**Class B & S  
autoclave**



## **Supersafe**

**Safety for your fluffy patients:** Class B and Class S autoclaves with the fractional vacuum procedure provide the safest sterilization of all wrapped and unwrapped instruments - regardless of their type and complexity.

Class B and S steam sterilization according to EN 13060

## **Superstrong**

**More than enough power for whatever you plan to do:** High loading capacities, fast cycle times from 16 minutes and effective vacuum drying ensure that your instruments are ready for use in no time.

Up to 6 kg on 5 extra long trays



## **Supersmart**

**The intuitive user interface with a large Smart-Touch Display makes everything easier:** From program selection to documentation. With the help of process monitoring, you can provide complete proof of a successful sterilization.

Documentation via USB, network or log printout\*

## **Superrobust**

**An autoclave should never fail:** We make no compromises when it comes to reliability and develop under the highest quality requirements at our headquarters in Berlin.

100 % quality - made in Germany



## The faithful companion for veterinary practices and clinics



*„With our solutions for instrument reprocessing, we offer the highest protection for our faithful friends: the pets!“*

Animalogic Veterinary Practice,  
Spain



*„Our top priority is our patients and their owners: that's why we rely on MELAG for reprocessing.“*

AniCura Small Animal Specialists,  
Germany



*„The system solution enables perfect workflows for time- and cost-saving reprocessing.“*

Clinica Veterinaria CMV,  
Italy

# Protects the environment. And your furry patients.

SteriHero Vet guarantees safe sterilization according to your needs with a wide range of programs.

But that's not all: the pioneering stand-alone device concept and the Power Save energy-saving function make SteriHero Vet the cleanest solution - for your budget and the environment.

Programs SteriHero Vet 23	Universal-B	Quick-S	Quick-B	Gentle-B	Prion-B
<b>Program specification</b>	134 °C, holding time: 5.5 min	134 °C, holding time: 3.5 min	134 °C, holding time: 5.5 min	121 °C, holding time: 20.5 min	134 °C, holding time: 20.5 min
<b>Operating times</b> (at minimum to maximum load) *	24 – 31 min	16 – 20 min	24 – 27 min	39 – 47 min	37 – 48 min
<b>Drying</b> (time controlled)	20 min	5 min	10 min	20 min	20 min

Programs SteriHero Vet 23 S	Universal-S	Quick-S	Gentle-S	Prion-S
<b>Program specification</b>	134 °C, holding time: 5.5 min	134 °C, holding time: 3.5 min	121 °C, holding time: 20.5 min	134 °C, holding time: 20.5 min
<b>Operating times</b> (at minimum to maximum load) *	20 – 25 min	16 – 20 min	36 – 38 min	35 – 40 min
<b>Drying</b> (time controlled)	25 min	5 min	25 min	25 min

\* Data for the Quick S program for unpacked loads and for all other programs for single-packed loads. Times may vary depending on load type and quantity.

Device / Order no.	Device type	Chamber size	Chamber volume	Load quantity	Device dimensions (W x H x D)	Weight	Power supply	Documentation interfaces
<b>SteriHero Vet 23</b> ME10123VB	Stand-alone class B autoclave (compatible with additional water treatment)	ø 25 cm x 45 cm long	23 liters	max. 6 kg instruments / 2 kg textiles	47 x 50 x 64 cm (device fits on a 60 cm table top)	49 kg	220 – 230 V / 50 – 60 Hz (2.100 Watt)	USB, Network
<b>SteriHero Vet 23 S</b> ME10123VS	Stand-alone class S autoclave (compatible with additional water treatment)	ø 25 cm x 45 cm long	23 liters	max. 5 kg instruments / 1 kg textiles	47 x 50 x 64 cm (device fits on a 60 cm table top)	47 kg	220 – 230 V / 50 – 60 Hz (2.100 Watt)	USB, Network

Rev.0-24/0040-01.24-EN-ME



**Save paper, protect the environment:**

In order to reduce paper, we provide you with electronic user documentation via our Download Center. Further information at: [www.sterihero.com](http://www.sterihero.com)

**MELAG**